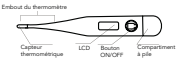
## BIOSŸNEX

Thermomètre digital

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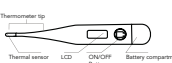
## BIOSŸNEX

Digital Thermometer

Please read the instructions carefully before using the thermometer for the first time, and keep these in a safe place. This product is intended for the measurement of human body temperature.

#### OPERATING INSTRUCTIONS

Before using, please disinfect the probe at first. To switch on, press the ON/OFF button next to the display; a short beep will sound, indicating that the thermometer is operational. At the same time the thermometer runs a self-check test, during which all the digital segments appear on the LCD.

During the reading, the current temperature is displayed continuously and the " C " or " F " symbol flashes. The measurement is completed when a constant temperature value has been reached. The temperature value is considered constant when the temperature rises less than 0.1 C within 4 seconds. As soon as the constant temperature value is reached, a beep will sound four times, and the " C " or " F " symbol will stop flashing. The highest temperature measured appears on the LCD. However, please note that this thermometer is a maximum thermometer, i.e. the displayed temperature can increase slightly if measurement continues after the beep. When the measurement is completed, please switch the thermometer off by pressing the ON/OFF button. After the temperature has been displayed, the thermometer will shut off automatically after 10 minutes.

#### MEMORY FUNCTION

Switch the thermometer off, then press the ON/OFF button for about 2 seconds. The last measured value with " C " or " F " will appear automatically on the LCD. This display is visible as long as the ON/OFF button is held down. The reading is only over-written when a new temperature value is recorded.

#### METHODS OF MEASURING TEMPERATURE

It is important to remember that the body temperature reading depends on the site where it is measured. For this reason, the measurement site must always be specified in order to ensure that a correct temper ature reading is recorded.

#### In the rectum(rectal)

This is the most accurate method from a medical point of view, because it comes closest to the core body temperature. The thermometer tip is inser ted carefully into the rectum for a maximum of 2 cm. The usual measuring time is approximately 40 to 60 seconds.

#### Under the arm(axillary)

Placing the thermometer in the armpit provides a measurement of surface temper ature that can fluctuate by around 0.5 C to 1.5 C from rectal temperature readings in adults. The usual measuring time for this method is approximately 50 to 70 seconds.

#### In the mouth(oral)

As a gener alrule, the oral temperature is 0.3 C to 0.8 C lower than the rectal temperature. To ensure that reading is as accurate as possible, place the thermometer tip to the left or right of the root of th e tongue. Place the thermometer tip under the tongue in one of the two heat pockets at the back, keep the mouth closed during the reading and breathe evenly through the nose. The usual measuring time is approximately 15 to 19 seconds.

#### CLEANING AND DISINFECTION

The best way to clean the thermometer tip is by applying a disinfectant (e.g. 70% ethyl alcohol) with a damp cloth. It should be disinfected before each use The Waterproof ther mometers can be immersed in liquid or lukewarm water for thorough cleaning and disinfection, but the Not Waterproof ther mometers can not be allowed.

#### SAFETY PRECAUTIONS

- Do not allow the device to come into contact with hot water.
- Do not expose to high temperatures or direct sunlight.
- Do not drop the thermometer. It is neither shock-proof nor impact-resistant.
- Do not modify this device without authorization of manufacturer
- Do not clean with thinners, petrol or benzene! •Only clean with water or disinfectant.
- Do not immerse the Waterproof thermometers under water 15cm for longer than 30 minutes..
- The thermometer contains small parts (battery, battery compartment) which can be swallowed by children. For this reason, do not leave the thermometer unattended in the hands of children.
- Avoid bending the Non-Flexible thermometer tip.
- Flexible thermometer, avoid bending the tip by more than 45 degrees
- If the ambient temperatu re is over 35 C or 95 F, dip the thermometer tip in cold water for approx.
- 5 to 10 seconds prior to measuring the temperature.
- Persistent fever, in particular in children, has to be treated by a doctor- please get in touch with your doctor!
- Do not use near strong electromagnetic fields, i.e. keep it away from any radio systems and mobile phones.

#### BATTERY REPLACEMENT

The battery is empty and needs replacing when the « ▲ » or « g » battery symbol appears on the right of the LCD. Remove the battery cover and replace it with a battery (preferably non-mercury) of the same type.

We advise you to remove the batteries if the device is not going to be used for a longer period of time.

- Type: maximum thermometer •Measurement range: (32.0– 42.9)°C
- Others:(32.0– 42.9)°C •Measurement accuracy: +/- 0.1° C(35.5° C–42.0° C)
- +/-0.2° C(for the remaining measurement range) •Storage/transportation temperature: (-25–55)°C<95%RH
- Ambient temperature during use: (5–35)°C<80%RH
- Ambient temperature during use: (5–35)°C<80%RH •Min Scale: 0.1° C
- Battery type: •Alkaline manganese battery, type LR41, 1.5V/or CR 1220, 3V
- service life minimum 100 hours under continuous operation.
- Weight •Approx. 10g •Shelf life: 3 years

This thermometer is initially calibrated at the time of manufacture.

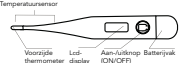
#### WARRANTY

This product is warranty for 2 year from the date of leave factory. Damage resulting from incorrec t use or abuse is not covered by the warranty. Battery and packaging are excluded from the warranty. Claims beyond this, including claims for damages, are excluded. If you find that the thermometer is defective and not in good function, please firstly check the battery before sending in for repair. The manufacturer will make available on request circuit diagrams, component part lists, descriptions, calibration instructions, or other information that will assist service personnel to repair those parts of device.e tongue. The thermometer tip must have constant contact with the tissue during the reading and be

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